

Ultra

3PLY Face Mask: STPM02

User Instructions

IMPORTANT: Keep *User Instructions* for reference



Step 1:

Hold mask by ear loops, taking care not to touch the front or back of the mask. Cover your nose and mouth and place the loops over your ears, ensuring the metal strip is on the bridge of your nose.



Step 2:

Squeeze the metal strip over the bridge of your nose so that it remains secure.



Step 3:

When the mask has been soiled, remove it by holding the ear loops.



Step 4:

Dispose of the mask in a bin.



Step 5:

Ensure you thoroughly wash your hands with soap and water after disposing of your mask.

- **PRECAUTIONS AND WARNINGS**

- Do not use mask if packaging is damaged
- Product is single use only; do not reuse and dispose of after use
- Use immediately after opening
- Only wear masks with soft side towards the face
- When wearing, ensure nose bridge metal strip is pressed tightly to avoid leakage
- Replace the mask if it is contaminated or damaged

- **TRANSPORT AND STORAGE**

- Store in well-ventilated room with relative humidity of no more than 80%
- Protect product during transport to avoid damage to package